
Instructions for Use and Care

Why ZQuiet® Works

Snoring is a vibration of soft tissue against the back of the throat. As we sleep, the tongue relaxes and partially closes the airway causing vibration, or snoring, as air passes in and out. The ZQuiet® device repositions your lower jaw slightly forward, improving airflow and reducing vibrations of the soft tissue in the throat.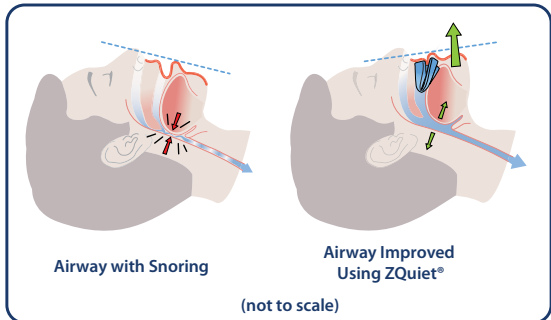

Indications for Use

The ZQuiet® anti-snoring device is intended for the treatment of nighttime snoring in adults.

Contraindications


- Do not use ZQuiet® if you have been diagnosed with Central Sleep Apnea.
- Do not use ZQuiet® if you have been diagnosed with severe respiratory disorders.
- Do not use ZQuiet® if you have loose teeth, loose dental work, orthodontia, full dentures or advanced periodontal disease.
- Do not use ZQuiet® if you have been diagnosed with temporomandibular joint disorders (TMD), experience clicking of the jaw and/or have pain in the jaw joints.
- Do not use ZQuiet® if you are under the age of 18 years.

Warnings


- Usage of ZQuiet® may cause:
- Tooth movement or changes in dental occlusion.
 - Gingival or dental soreness.
 - Pain or soreness to the temporomandibular joint.
 - Obstruction of oral breathing.
 - Excessive salivation.
 - Difficulty with speech.

Instructions

- Clean the device prior to initial use. Do not use if any parts are damaged or missing.
- Clean your teeth properly before inserting your appliance to prevent tooth decay and periodontal problems.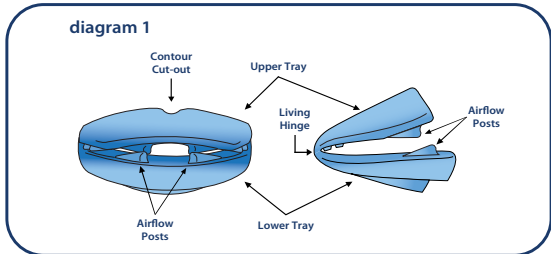

- Insert the device with the upper tray on top towards the back of your throat (see diagram 2). The contoured cutout located at the center-front (diagram 1) easily identifies the upper tray. You should notice that the lower jaw moves slightly forward of its normal position.

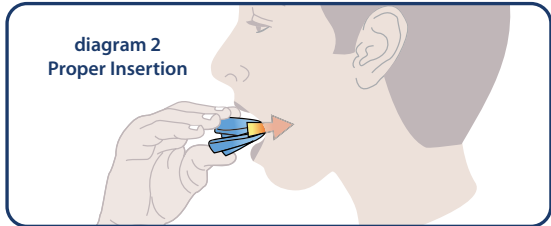- In the morning, remove your ZQuiet® device and follow the easy cleaning instructions.

Possible Side Effects

- You may experience some of the following reactions to ZQuiet®. These are normal and should disappear with continued use:
- You may feel pressure against your front teeth.
 - You may experience a gagging sensation or urge to cough.
 - You may experience increased salivation.
 - ZQuiet® may be ejected or inadvertently removed from the mouth during sleep.
 - You may experience a sensation of bite change.
- If you experience sharp or persistent pain lasting more than two hours after use, you should discontinue use and contact your dentist.

Cleaning Instructions

- ZQuiet® is for single patient use only and should be cleaned after each use to keep it free of harmful germs and bacteria. To clean the ZQuiet® Mouthpiece, follow these simple instructions:
- Using a toothbrush, scrub all surfaces with very warm water and a small amount of mild soap or detergent.
 - Rinse thoroughly with clean warm water to remove all soap.
 - After rinsing, shake off all excess water and pat dry with a soft clean towel.
 - Store ZQuiet® at room temperature in the supplied case.

Comfort Tips

- You may experience an adjustment period with ZQuiet®. Your jaw muscles may experience temporary soreness, like the soreness that may happen to muscles when trying a new exercise.
- First-time users can soak the device in hot tap water for one minute to slightly soften the device prior to inserting in mouth.
 - During the initial break-in period it is recommended to wear the device for short periods of time during waking hours. Insert ZQuiet® for 20-30 minutes while reading or watching TV.
 - You can trim ZQuiet® with small scissors to improve fit and comfort. After trimming, use a fingernail file or emery board to smooth any rough edges and avoid irritation to the gums.
- Upon waking in the morning, the lower jaw may continue to feel more forward than usual. This sensation should go away slowly during the next hour or so. If you experience sharp or persistent pain lasting more than two hours after use, you should discontinue use and contact your dentist.

Manufactured by:

ZQuiet®
5247 Shelburne Rd., Suite 204
Shelburne, VT 05482
U.S.A.

Copyright © 2012 Sleeping Well, LLC. All Rights Reserved
ZQuiet® is a Registered Trademark of Sleeping Well, LLC.
Patents Pending
ZQI-FA 01.12

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Copyright © 2012 Sleeping Well, LLC. Alle Rechte vorbehalten
Copyright © 2012 Sleeping Well, LLC. Tous droits réservés.
Copyright © 2012 Sleeping Well, LLC. Alle rechten voorbehouden
Copyright © 2012 Sleeping Well, LLC. Tutti i diritti riservati
Copyright © 2012 Sleeping Well, LLC. Všechna práva vyhrazena. ZQuiet® je registrovaná ochranná známka společnosti Sleeping Well, LLC. Patenty ve schvalovacím fízení. ZQI-FA 01.12
Copyright © 2012 Sleeping Well, LLC. Wszelkie prawa zastrzeżone ZQuiet® je registrovaná ochranná známka spoločnosti Sleeping Well, LLC. Patenty zjogzbozny ZQI-FA 01.12